

Comfortland

Mesh Shoulder Immobilizer

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THIS DEVICE.

INDICATIONS

The Comfortland mesh shoulder immobilizer provides immobilization and support for rotator cuff injuries, shoulder instabilities, and anterior dislocations of the G-H joint. Effective in relieving strain on the shoulder by suspending the arm.

CAUTION

The application instructions listed above are intended for the use of qualified professionals only. It is not recommended for patients to attempt to apply or adjust this device unless otherwise instructed to do so by an orthopedic professional. Any questions regarding further adjustment or frequency/ duration of wear should be directed to your orthopedic professional, as they know the individual condition for which this device was prescribed.

CLEANING INSTRUCTIONS

Hand wash in cold water with mild detergent. Allow all parts to air dry before re-assembly. Do not dry clean, iron, or bleach any part of this brace, as it could cause damage and void the manufacturer's warranty.

NOTICE: While every effort has been made to obtain maximum strength, function, and durability, there is no guarantee that injury will be prevented through the use of this device.

Comfortland Medical, Inc.
709 AO Smith Rd.
Mebane, NC 27302
T 866.277.3135 | F 866.277.4412
www.comfortlandmed.com

APPLICATION

1) Place arm in the shoulder sling (Fig. A), making sure the elbow fits flush in the seam of the sling, and that the hand hangs slightly out of the opening on the front of the sling (Fig. B).



2) Gently bring the strap over the head, so that it rests on the opposite shoulder, and drapes across the chest (Fig. C)



3) Unfasten the strap from itself using the Velcro provided, and adjust so that the arm is suspended in a bent position, and little to no strain is put on the shoulder (Fig. D).

