G-FORCE BRACES LUNA ADJUSTABLE

G-FORCE BRACES LUNA

G-FORCE

G-FORCE BRACES
291 SHASTA ST., MANCHESTER, NH 03103
888-248-9924 • gforcebraces.com

ENGLISH

ESPAÑOL

DEUTSCH

ITALIANO

FRANÇAIS

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A

SIZE ADJUSTMENT

A. To adjust the size of the G-Force Braces Luna, peel up the inside end of the side panel on each side. Adjust to achieve a proper fit and reattach.
B. Size ranges are only guidelines. Adjust as needed to achieve a snug fit without too much overlap.

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47 – 54 in

PREPARATION

Place the pull tabs 2 inches (5 cm) from the plastic in the back of the G-Force Braces Luna brace. Pull the ends of the brace away from each other ensuring that it is fully extended.

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LATERAL PANEL SIZING

Add or remove panels for appropriate sizing.

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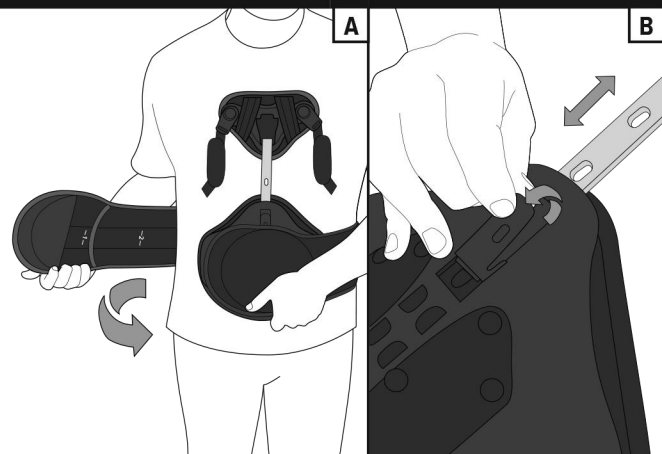
ITALIANO

FRANÇAIS

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PLACEMENT & ADJUSTMENT
A. Center the back panel as low as possible. Slide abdominal panel and sternal pad sideways as needed to ensure brace and panel are at patient's midline.
B. To determine sternal pad height, pull button back and slide aluminum strut so that the button locks into the appropriate hole.

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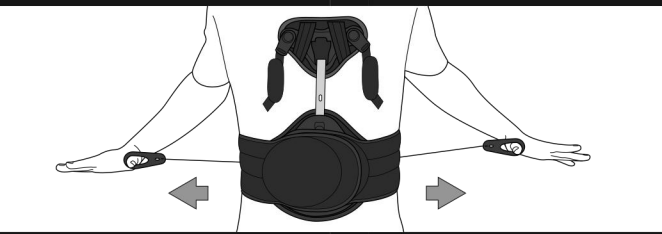
TIGHTENING
 Slip your thumbs through the holes in the pull tabs. Pull away from your body. The left pull tab controls lower compression and the right pull tab controls the upper compression. Adjust the compression as needed. When tightened, secure the pull tabs to the loop in front of the brace.

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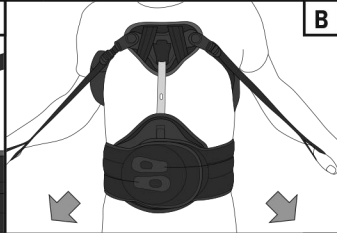
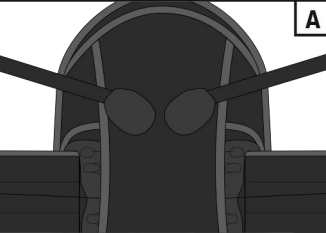
SHOULDER STRAPS
A. Attach control straps to back panel. Straps can be positioned to fit under the arms or over the shoulders.
B. Control straps and sternal pad should fit securely to patient's anatomy. Pull straps to tighten as needed.
C. Torso control strap setting.
D. Over the shoulder setting.
NOTE: If needed, the support strut can be bent to ensure a proper fit.

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